



A **FASTER, EASIER, SAFER**
APPROACH TO
QUALITY CONTROL

LyfoCultsTM
+++ PLUS

ATCC Licensed
Derivative[®]



LyfoCultsTM +++ PLUS

EASY TO USE ALL-IN-ONE DESIGN

- Glass free for your safety
- Built-in inoculation device for your convenience
- Hydrates immediately and requires no preincubation



ATCC Licensed
Derivative[®]

ATCC Licensed Derivative[®] Program (LDP) Guarantee

ATCC (American Type Culture Collection) is the world's leading reference organization with over 80 years of experience in the acquisition, authentication, production, preservation development and distribution of reference organisms.

bioMérieux participates in the ATCC Licensed Derivative Program (LDP) which is an international standard for monitoring the quality of organisms that are derived from ATCC materials and used in QC processes.

By joining the program, bioMérieux agrees to maintain the integrity of the organisms and verify their quality (identity, purity and viability of the organism).



Tear open foil pouch at notch and remove device. Remove label from the outside of the pouch.



Place the label from the pouch onto the culture medium.



Unscrew applicator from vial. Holding applicator at a slight angle to vial, activate ampoule by squeezing button firmly.



With the applicator at a slight angle to the vial, squeeze the ampoule dispensing the hydration fluid into vial. Then re-insert applicator into vial and gently mix from side-to-side. Do not invert.



Using ampoule as a transfer pipette, place two drops of resulting suspension on culture medium.



Streak the medium for isolation and incubate in appropriate atmosphere.

bioMérieux S.A.
69280 Marcy l'Etoile
France
Tel.: 33 (0)4 78 87 20 00
Fax: 33 (0)4 78 87 20 90

www.biomerieux.com
www.biomerieux-diagnostics.com/lyfocultsplus

